## FLEXIFIT™ 405 NASAL MASK PATIENT INSTRUCTIONS FOR USE

THE FLEXIFIT™ 405 NASAL MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT.

### FITTING YOUR MASK

- Remove plastic cover from Silicone Seal (C).
- Holding your mask in one hand, spread the Headgear (K) with your free hand, just enough to fit comfortably over your head.
- Lightly hold the mask against your face, covering the nose, then slowly pull the Headgear over your head.
- Attach the Headgear to the Glider™ Strap (E).
- Pull the lower straps as low as possible down the back of your neck. Gently tighten straps to remove slack. Connect to CPAP/Bi-Level machine at prescribed pressure.

- If necessary, tighten the lower straps to eliminate any leaks (avoid overtightening the straps).
- Use the Silicone Seal that provides the best fit for you. Small and Large are both provided with your mask.
- When you need to remove your mask, unhook the Headgear from the Glider™ Strap.

### FITTING TIPS

- Forehead pads should rest gently on the forehead without top straps being pulled too tight.
- If there are any leaks after gently tightening the top and lower straps, pull the mask slightly out from the face while straps are still attached. The FlexiFit™ 405 Nasal Mask may be used with, or without the Diffuser and Diffuser Cap (G).

### CLEANING YOUR MASK

To ensure the continuing safety and comfort of your FlexiFit™ 405 Nasal Mask:

- BEFORE FIRST USE:**
  - Hand-wash the Headgear (K) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes then rinse in fresh water and allow to dry out of direct sunlight.
- BEFORE EACH USE:**
  - Inspect the mask for deterioration. Do not use if damaged.
- AFTER EACH USE:**
  - Disassemble the Silicone Seal (C), Foam Cushion (D), Elbow (B) and the Headgear (K) from the Mask Base (A).
  - Wash the Silicone Seal and Mask Base in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water.
  - Wipe the Foam Cushion with a damp cloth. (Do not immerse the Foam Cushion in water).
  - Leave all parts to dry out of direct sun light before reassembling. The Diffuser (F) should not be washed or rinsed. Simply replace it with a new Diffuser if it becomes noisy or deteriorates. It should last two to three months. Small dots may appear on the Diffuser - these do not affect its performance. The FlexiFit™ 405 Nasal Mask may be used with, or without the Diffuser and Diffuser Cap (G).
- EVERY 7 DAYS:**
  - Hand wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to air dry out of direct sunlight.

### CAUTIONS

- Do not soak the mask for longer than 10 minutes.
- Do not wash the mask in dishwasher.
- Do not clean the mask with products containing alcohol, antibacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your FlexiFit™ 405 Nasal Mask weakens or cracks, discontinue use and seek replacement immediately.

### ASSEMBLING YOUR MASK

Your FlexiFit™ 405 Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask, use the following instructions for reassembly.

Refer to the mask assembly diagram.

- Place the Foam Cushion (D) into the Mask Base (A).
- Starting at the top of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom of the mask.
- Push the Elbow (B) into the Mask Base.
- Lay the Headgear (K) flat, with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (I) and the Glider™ Strap (E). This can be done without undoing the VELCRO® brand tabs\*, by sliding the Headgear into the slots.

\*VELCRO® is a registered trademark of Velcro Industries BV.

### WARRANTY AND DISPOSAL

#### WARRANTY STATEMENT

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in <http://www.fphcare.com/osa/productwarranty.asp>

#### DISPOSAL

This device does not contain any hazardous substances and may be disposed of with general waste.

### WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly.

**Explanation of Warning:** CPAP and Bi-Level machines are intended to be used with special masks which have exhaust flow holes to allow continuous flow of air out of the mask. When the CPAP/Bi-Level machine is turned on and functioning properly, new air from the CPAP/Bi-Level machine flushes the exhaled air out through the mask exhaust flow holes. However, when the CPAP/Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.

- At low CPAP/Bi-Level pressures the flow through the exhaust flow holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Do not block the gas exhaust flow holes or use alternative Diffusers, other than those manufactured by Fisher & Paykel Healthcare. Before use, always ensure that gas is flowing out the exhaust flow holes or through the Diffuser (if it is being used).
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If re-using the FlexiFit™ 405 Nasal Mask between different patients, refer to the Multi-Patient Cleaning Instructions for the FlexiFit™ 405 Nasal Mask.

### CAUTIONS

- Use the mask for its intended use only as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of, a physician.

The FlexiFit™ 405 Nasal Mask and accessories are Latex free. Many people find CPAP therapy causes dryness of the nose and throat. Heated humidification can substantially reduce these symptoms.

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## OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or supplemental oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

- Connect the large end of the Connector (T) to the Swivel (J) on the mask and the small end of the Connector to the Breathing Tube (R).
- Open the Port Cap(s) (P) and place the pressure and/or oxygen tube (S) firmly over the port(s) (Q).

(The two ports are identical; either Port may be used for oxygen or pressure measurement. Only remove the Port Cap from ports that are being used.)

- Note:** At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

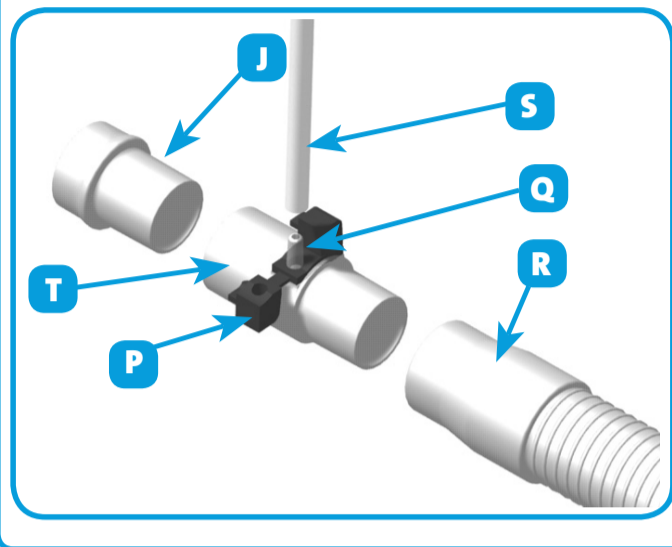
- Warning:** If oxygen is used with this CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.

**Explanation of Warning:** When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.

- Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.

- Warning:** Do not use Polyvinyl Chloride (PVC) pressure and/or oxygen tubing. Use silicone tubing instead.

**Explanation of Warning:** Contact with PVC can cause premature cracking of ports.


## TECHNICAL SPECIFICATIONS

### EXHAUST FLOW

Nasal masks for CPAP and Bi-Level have exhaust flow holes to expel the air that you exhale from the mask. The FlexiFit™ 405 Nasal Mask has a unique Diffuser system that is designed to diffuse the air that is expelled from the mask. This reduces the draft. The Diffuser does not significantly change the amount of air flowing from the mask, therefore the mask can be used with or without the Diffuser.

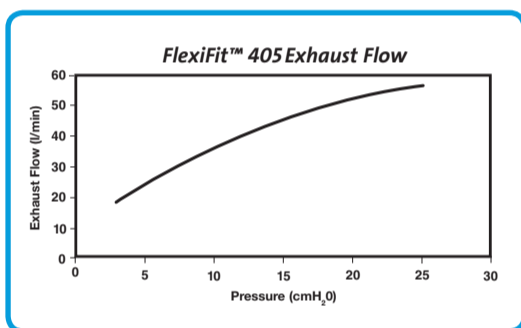PRESSURE (cmH <sub>2</sub> O)	3	5	7	9	11	13	15	17	19	21	23	25
FLOW (L/MIN)	18	24	29	33	37	41	44	47	50	53	55	58

### RESISTANCE TO FLOW

Pressure drop through mask @ 50 l/min is 0.1 ± 0.1 cmH<sub>2</sub>O  
Pressure drop through mask @ 100 l/min is 0.5 ± 0.1 cmH<sub>2</sub>O

### MASK DEAD SPACE

88cc

### OPERATING CONDITIONS

- Operating pressure range is 3-25 cm H<sub>2</sub>O.
- Operating temperature range is 5-50°C (40-120°F).

### SOUND

The Sound Power Level of the mask is 28 dBA, with uncertainty 2.5 dBA.  
The Sound Pressure Level of the mask is 22 dBA, with uncertainty 2.5 dBA.

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© 2010 Fisher & Paykel Healthcare Limited

Fisher & Paykel  
HEALTHCARE

## MULTI-PATIENT CLEANING INSTRUCTIONS

Your FlexiFit™ 405 Nasal Mask is provided patient-ready. To reuse between patients please follow the guidelines below.

### CLEANING / DESOILING

- Before disinfection, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

### HIGH-LEVEL DISINFECTION PARAMETERS

NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

MASK BASE	<b>AUTOCLAVING</b> Autoclave 15 minutes' exposure at 121°C (250°F). <b>OR</b> <b>THERMAL DISINFECTION</b> Soak parts in hot-water bath for 30 minutes at 70°C (158°F). <b>OR</b> <b>STERRAD® 100S</b> Follow manufacturer's instructions for Sterrad® 100S sterilization system. <b>OR</b> <b>CIDEX® OPA</b> Follow manufacturer's instructions for high-level disinfection. Immerse for 12 minutes at 20°C (68°F). Rinse thoroughly in potable water. <b>OR</b> <b>SPOROX® II</b> Follow manufacturer's instructions for high-level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
-----------	--

HEADGEAR	Hand-wash the Headgear in pure soap dissolved in lukewarm water. Dry out of direct sunlight.
----------	--

FOAM CUSHION	Replace with new part.
--------------	------------------------

DIFFUSER DIFFUSER COVER	If continuing use with the Diffuser, replace with a new part.
----------------------------	---

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THE FLEXIFIT™ 405 NASAL MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.

DISCLAIMER: IT IS THE USER'S RESPONSIBILITY TO QUALIFY ANY DEVIATIONS FROM THE RECOMMENDED METHOD OF REPROCESSING.

CONTACT YOUR LOCAL HOME CARE PROVIDER FOR ADDITIONAL INFORMATION.

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This product is covered by one or more of the following patents or patent applications in the corresponding countries listed below.

USA: US 6615834, US 6666283, US 6789541, US 6791926, US 6951218, US 7017576, US 7308895  
Australia: AU 784321, AU 785331, AU 2002302002, AU 2003281188, AU 2004294873, AU 783749  
Europe: EP 1163924 (GB, IT, ES), EP 1905475, EP 1306098, EP 1289590, EP 1521613, EP 1270593, EP 1163923 (IT, GB, ES)