

## Battery Instructions

Thank you for purchasing your **Transcend P4 or P8 Battery**.

### Use instructions:

1. Fully charge the battery before first use. When the green LED light on the battery turns green, the battery is fully charged.
2. Charge the battery using your Transcend Universal Power Supply, Transcend DC Mobile Power Adaptor, or Transcend Portable Solar Battery Charger as shown below.



3. Insert the barrel connector plug from the battery into the Transcend device as shown:



The following table shows the operating and storage conditions compatible with your battery:

Operating range	Storage range
32° to 113 F (0° to 45° C)	-4° to 140° F (-20° to 60° C)
10%-80% relative humidity, non-condensing	10%-90% relative humidity, non-condensing

**Note:** Leaks in the system airway circuit or use at non-pressurized altitudes above 8,000 ft (2,500 m) may affect hours of battery operation. [Note: Aircraft are pressurized. Use in aircraft will not affect hours of battery operation.]

Please refer to your Transcend User Manual for more information. For more information contact your home medical equipment dealer or call Somnetics customer service at 877-621-9626.


**CPAPMAN.COM**  
 BEST SERVICE SINCE 1997  
[Click Here To Order Today](#)

RESPSHOP

**Phone:** 866-936-3754

**Fax:** 866-936-3730