

CHECK LIST

Locate the following items in your X-PLOR® box to get started:



X-PLOR Unit



Carrying Bag



Replaceable Cartridge
(found in Mylar bag)



AC Power Supply



DC Power Supply



Handle
(Attaches to Carrying Bag)



Shoulder Strap
(Attaches to Carrying Bag)



8-Cell Battery



TRAVEL WITH X-PLOR

The X-PLOR lets you travel with ease. It's perfect for carrying on buses, trains, planes, and cruise ships! Now with X-PLOR supplemental oxygen and direct-charge batteries, exploring has never been so easy.

X-PLOR IS FAA COMPLIANT.



XPLORoxygen.com | (877)-GO-XPLOR

©2023 Belluscura LLC. All Rights Reserved.



Quick
START GUIDE

belluscura® | XPLORoxygen.com

Quick START GUIDE

DO NOT TURN ON THE X-PLOR UNTIL YOU REACH STEP 4.
It is quick and easy to get your X-PLOR started, but please read the entire User Manual before using.



REMOVE DUST CAP & INSERT CARTRIDGE

- Remove cartridge from mylar bag.
- Remove the dust cap from the cartridge.
- Then insert the cartridge all the way into the unit so the handle lays flat.



INSERT INTO BAG & ATTACH THE BATTERY

- Insert your X-PLOR into the carry bag. When the X-PLOR is correctly inserted into the bag, the LCD screen is seen through the plastic window and the air intake and exhaust are aligned with mesh panels.
- To attach the battery, slide it on until the latch is in the locked position and zip the bag.



CHARGE THE BATTERY

- Place in a well-ventilated area to fully charge the battery before using your X-PLOR, which can take up to 6 hours for the 8-cell battery.
- To charge, insert the AC power connector into the battery's power receptacle and the other end into an electrical outlet.




TURN ON & SET OXYGEN FLOW

- Press the POWER button for 2 seconds to turn on your X-PLOR.
- When powered on, position the unit upright to warm-up (Warm-up time is less than 2 minutes.)
- Use the UP/DOWN ARROWS to set the oxygen flow level, as specified in your oxygen prescription.



NASAL CANNULA

- Cannulas must be purchased separately.
- Connect the nasal cannula to the oxygen output nozzle located on top of the X-PLOR.
- Place the other end in your nostrils. Oxygen will be delivered through the nasal cannula with each breath.
- Remember to breathe through your nose as normal.

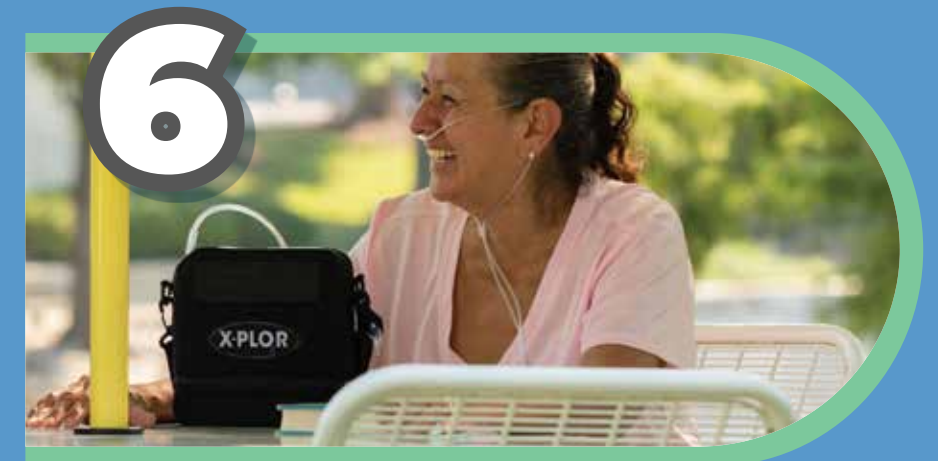

GO X-PLOR!

- Attach the shoulder strap and handle to your X-PLOR device and enjoy your new found mobility!

**LIVE LIFE. BREATHE EASY.
GO X-PLOR.**