

Installing Your PR System One Humidifier Adapter



CPAPMAN.COM
BEST SERVICE SINCE 1997


[Click Here To Order Today](#)

Remove the original reservoir from your CPAP humidifier.

Determine which of the two new reservoir lids match your original one and install the new matching lid onto your new humidifier reservoir. The extra lid can be discarded.

Insert the new reservoir into your CPAP unit like you normally would.


Disconnect the SoClean fitting along with its short hose from the the main hose, as shown in [Figure A](#).

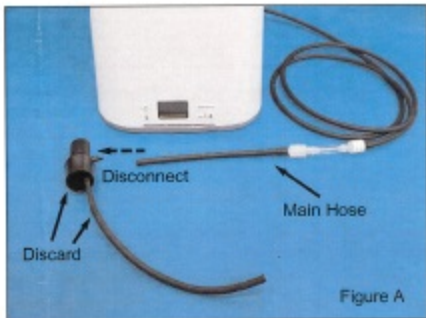

Connect the main hose to the hose coming from your new reservoir by pushing the hose end onto the plastic stem of the short hose as shown in [Figure B](#).

Hint: a little water or dish soap on the end of the plastic connector can help it slide on easier.

Your humidifier reservoir hose is now connected using an easy-disconnect coupling. This coupling will allow you to quickly separate your hose, allowing you to easily remove your reservoir for maintenance.

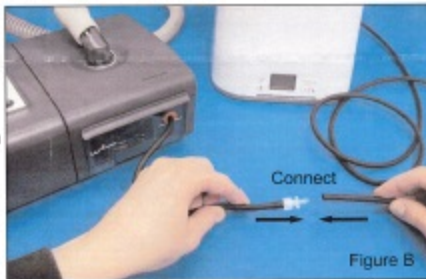

To use the easy-disconnect coupling, hold both sides of the plastic coupling, unscrew and pull apart as shown in [Figure C](#). The hose will separate at the coupling. Reverse this action to reconnect the hose.

IMPORTANT: You must **reconnect** your hose before operating your CPAP. If you fail to do this, the water in your humidifier will be pushed out through the hose and flooding will result. Better Rest Solutions is not responsible for misuse of its equipment. Any misuse may void the warranty.

