

CLEANING INSTRUCTIONS



CPAPMAN.COM
BEST SERVICE SINCE 1997

[Click Here To Order Today](#)

AFTER EACH USE: Rinse the mouthpiece in cold water and wipe the outer portion of the pillows with a damp cloth.

WEEKLY: Fill your sink with cold water and add a mild liquid soap such as unscented Ivory liquid. Disassemble the mask and soak all parts including headgear for no longer than 10 minutes. Remove and rinse in cold water then allow the parts to air dry. Reassemble the mask before your next use.

WARNING: For best results do not use any cleansers that contain alcohol, bleach, moisturizers, anti-bacterial agents, or scents.