



[Click Here To Order Today](#)

Fitting Guide



- Prior to fitting the mask, remove the elbow from the mask by pressing the squeeze-tabs.



- Connect the elbow to the device air tubing.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, loop the headgear into the lower hooks on the mask frame.



- Unfasten the straps, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the straps.



- Repeat step 5 with the lower headgear straps.



- Attach the combined elbow assembly and air tubing to the mask by pressing the squeeze-tabs and pushing into mask frame.
- Turn on the airflow device.

Pair the Mirage FX with ResMed's S9™ for an exceptionally quiet, comfortable system that fits your patients' lifestyles.



Mirage™ FX
NASAL MASK

ResMed.com