

Follow these instructions to properly position your EasyFit headgear and mask. The device must be turned on in order for the mask to seal to face properly. There should be minimal or no leaks.

NOTE: Refer to instruction manual for more detailed instructions.



STEP 1



Unlock one of the lower headgear clips on the mask.

STEP 4




The forehead support and the mask should fit securely against the face. The forehead support should be adjusted until mask and forehead support are parallel.

STEP 2




Guide headgear over the head, making sure straps rest on the top of the head and cross in the back. Clip loose end back onto mask.

STEP 5

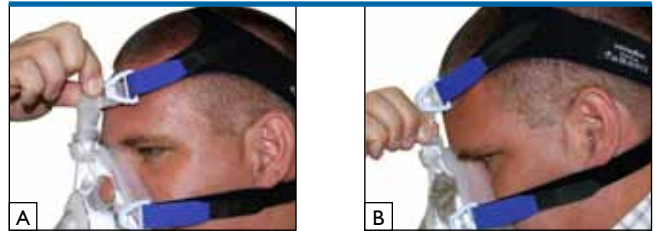

To adjust forehead support, squeeze base of the coarse adjustment component and position in slots (A). For additional adjustments, squeeze top of coarse adjustment component to move in, out, up, and down (B).

STEP 3

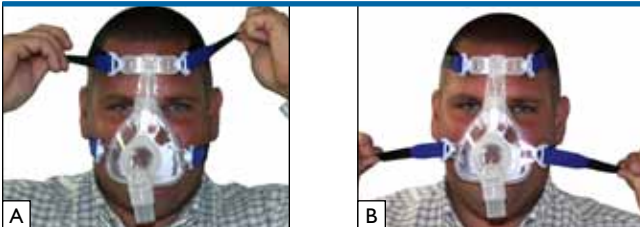

Adjust velcro straps so that the mask fits firmly, but not too tightly on head, leaving room for an index finger to fit comfortably between the forehead and straps on each side.

STEP 6

After mask is fitted, test that it is in the proper position by setting the device to the prescribed pressure. There should be minimal or no leaks.

Anti-Asphyxia Valve



Closed

Open

Ensure the round flap on the anti-asphyxia valve is mobile when air is on. This will prevent suffocation in the event of a power failure while CPAP is in use.