RESMED

Air Tubing User Instructions

English • Deutsch • Français • Nederlands • Svenska • Dansk • Norsk • Suomi

CPAPMAN.COM
BEST SERVICE SINCE 1997

Click Here To Order Today

A

B

C

D

Air Tubing User Instructions**Connecting the air tubing to the S9**

Refer to illustration A.

- ✓ Take one end of the air tubing and hold the cuff by the finger grips.
- ✓ Connect the air tubing firmly onto the air outlet.
- ✗ Do not hold the air tubing by the tube or twist it.

Disconnecting the air tubing from the S9

Refer to illustration B.

- ✓ Hold the cuff of the air tubing by the finger grips.
- ✓ Gently pull the cuff away from the air outlet and to the side.
- ✗ Do not pull on the tube itself or twist it.

Connecting the air tubing to the mask

Refer to illustration C.

- ✓ Hold the cuff of the air tubing by the finger grips and the swivel of the mask.
- ✓ Connect the free end of the air tubing to the swivel.
- ✗ Do not hold the air tubing by the tube or twist it.

Disconnecting the air tubing from the mask

Refer to illustration D.

- ✓ Hold the cuff of the air tubing by the finger grips and the swivel of the mask.
- ✓ Gently pull the cuff away from the swivel as shown.
- ✗ Do not pull on the tube itself or twist it.

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Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. **EC|REP** ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide.

S9 is a trademark of ResMed Ltd and registered in U.S. Patent and Trademark Office.

For patent information, see www.resmed.com/ip.

© 2012 ResMed Ltd. 368779/1 2012-01