

Designed **with care**



As the leader in mask technology, ResMed has partnered with pediatric health experts to bring you Pixi™ – a pediatric mask designed with children and their families in mind.

A mask designed specifically for children

Uniquely constructed using the anthropometric data of children aged two and older, the Pixi pediatric mask is not merely a scaled down adult mask.

Taking into account the bone structure, facial characteristics and skin sensitivity of children, the mask is designed to minimize pressure on soft faces.

The Pixi pediatric mask is made with both the child and their caregiver in mind.

Every feature is carefully considered to address and **improve therapy comfort, acceptance and overall experience for everyone involved.**

Confident that their child is being treated effectively, caregivers can be free to focus on emotionally supporting their child on their journey.



Pixi™
PEDIATRIC MASK



Technology highlight

Headgear designed especially for children*

- **Three points of adjustment**—the mask can be adjusted at the top, side and bottom straps, providing a greater range of adjustment to cater to children of various ages.
- **Out of sight**—the headgear is designed to sit away from the child's eyes and ears to reduce irritation and obstruction.
- **Clear line of sight**—not only can the child see clearly while wearing the mask, but caregivers are also reassured and comforted by being able to see their child's eyes and more of their face.

**Designed using the specific head and facial anthropometric data of children.*

Optional pressure port with tubing
not attached to the mask,
packed separately

Soft, comfortable cushion
single-wall, thin silicone
cushion minimizes pressure
on children's soft faces,
without compromising seal

Two tubing positions
air tubing can be moved
to either side of the cushion
depending on child's
sleeping position

Spring-flex lightweight tube
stretches and bends easily, so it won't
get caught or drag as the child moves
around during sleep

**Comfortable Breathe-O-Prene™
headgear materials**
lightly colored for a more reassuring,
unobtrusive appearance on the child's face

Side-facing backstrap
ergonomically designed from
the fitting adult's point of view

Quick-release latch
makes it easy to take the mask off,
while retaining desired headgear settings

Emergency latch
releases the latch even faster
in an emergency, by pulling firmly
on the lower headgear strap



Fitting Guide



Pixi Hint: If fitting the mask for the first time, we recommend the child be sitting in an upright position to ensure correct fit.



- Hold the mask gently against the child's face, checking that the nose is fully enclosed by the cushion and the mouth is clear.



- Pull the headgear over the head and move the middle strap down until it sits across the middle of the back of the head.



- Keeping your hand on the frame to maintain correct headgear position, bring the lower headgear strap behind the back of the neck.



- Attach the lower headgear strap by holding the tab and hooking the quick-release clip onto the mask.



- If necessary, adjust the straps by removing the Velcro® and gently tightening the headgear straps until comfortable. Do not overtighten.
- For optimal results, adjust the straps starting at the bottom strap and work upwards.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

Tip



- When fully fitted, ensure the top strap is not overtightened and is aligned with the top of the ears; the middle strap should sit across the back of the head aligned with the bottom of the ears.



Pixi™
PEDIATRIC MASK

ResMed.com


Components Guide

PRODUCT CODES



Complete Pixi system **61030**

Frame system (no headgear) **61035**

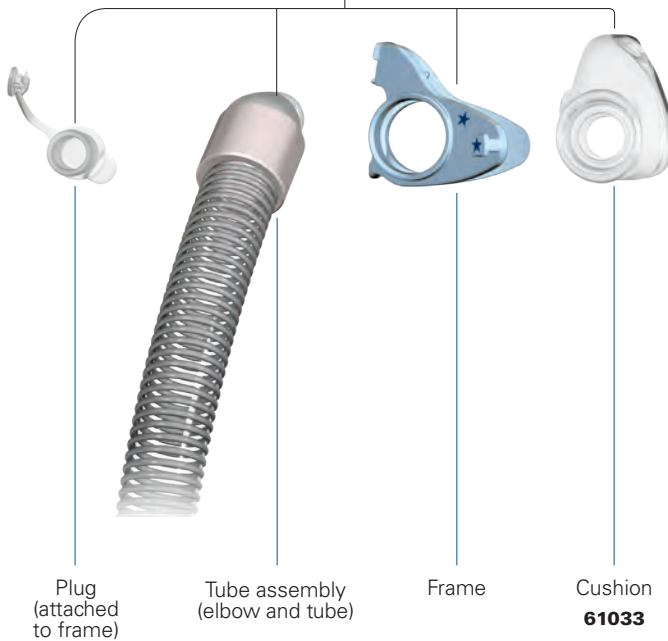

Plug
(attached
to frame)

Tube assembly
(elbow and tube)

Frame

Cushion
61033



Headgear
61034

 For details, refer to the **Pixi Pediatric Mask User Guide.**

REIMBURSEMENT CODES

HCPCS	Code Descriptor
A7034	Nasal mask used with positive airway pressure device, 1 per 3 months
A7032	Replacement cushion for nasal mask, 2 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

Combine the Pixi mask with the latest Stellar 100 or VPAP devices for a therapy system that delivers greater comfort, reliability and effective care.



Stellar™ 100



Pixi Mask



Pixi™
PEDIATRIC MASK

ResMed.com